

## PlasmaFlow DVT Sleeves

*Wear for 30-minutes, 1-2x daily*

### HOW TO USE

1. Unbox and plug charger into the wall.
2. Plug the (2) split leads into the corresponding port on the DVT sleeves, so they charge simultaneously.
3. When charging, the display area will blink **RED**.  
When fully charged, it will display **BLUE**.
4. Once charged and unit displays the solid **BLUE** light, simply place around the calves so the **RED** power button is facing upward on the inside of the left leg, and the outside of the right leg.
5. Hold down the power button for **3-seconds** and release once unit illuminates **BLUE**.
6. Each sleeve will automatically inflate to 55 mm/Hg (millimeters of Mercury) then deflate. 50-seconds will elapse and then the process repeats. Set a timer and wear for **30-minutes**, then hold down power button for 3-seconds to shut off.
7. When not in use, recharge.



**TROUBLE SHOOTING****Low Battery:**

When battery is low, **RED** light will flash, **E2** will display on left side of LED screen, **LO** will display on right side of LED screen. Recharge. Unit can be used while directly plugged into charger.

**Error Code E1/Sleeves Not Inflating:**

Make sure the sleeves are wrapped tight enough around calves. They must be secured tight enough for unit to function and properly inflate/deflate. Tighten sleeves, power off and power back on.